*No.1 choice
of hospitals
& mothers*

Swing maxi™

EN Instructions for use

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Thank you for choosing the Swing maxi breast pump. Breast milk is the natural choice for your baby, but there are reasons and situations where breastfeeding is not possible. For over 50 years, it has been Medela's conviction that these mothers and babies can be supported. In close co-operation with breastfeeding experts, Medela, one of the market leaders, has researched and developed breast pumps that perfectly meet breastfeeding mothers needs. For mothers who insist on the best for their babies and themselves. Because you care.

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13. EMC/Technical description

The Swing maxi breast pump needs special precautions regarding EMC (Electromagnetic compatibility) and needs to be installed and put into service according to the EMC information provided in these instructions for use.

Wireless communications equipment such as wireless home network devices, mobile phones, cordless telephones and their base stations, walkie-talkies can affect the electric breast pump Swing maxi and should be kept at least a distance 1.0 m away from the equipment.

Electromagnetic Compatibility (EMC, IEC 60601-1-2:2007, Table 1)

Electromagnetic emissions		
The electric breast pump Swing maxi is intended for use in the electromagnetic environment specified below. The customer or the user of the electric breast pump Swing maxi should assure that it is used in such an environment.		
Emission tests	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The electric breast pump Swing maxi uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment. The electric breast pump Swing maxi is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
RF emissions CISPR 11	Class B	
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Not applicable ^a	
^a No significant voltage changes		

 **Warning** – The electric breast pump Swing maxi should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, the electric breast pump Swing maxi should be observed to verify normal operation in the configuration in which it will be used.

Electromagnetic Compatibility (EMC, IEC 60601-1-2:2007, Table 2)

Electromagnetic immunity			
The electric breast pump Swing maxi is intended for use in the electromagnetic environment specified below. The customer or the user of the electric breast pump Swing maxi should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 6 kV contact ± 8 kV air	± 6 kV contact ± 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient / burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input / output lines	± 2 kV for power supply lines Not applicable ^a	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV line(s) to line(s) ± 2 kV line(s) to earth	± 1 kV differential mode Not applicable ^b	Mains power quality should be that of a typical commercial or hospital environment.
^a No input/output lines			
^b No earth, class II equipment			

13. EMC/Technical description

Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5 % U_T (>95 % dip in U_T) for 0,5 cycle 40 % U_T (60 % dip in U_T) for 5 cycles 70 % U_T (30 % dip in U_T) for 25 cycles <5 % U_T (>95 % dip in U_T) for 5 s	<5 % U_T (>95 % dip in U_T) for 0,5 cycle 40 % U_T (60 % dip in U_T) for 5 cycles 70 % U_T (30 % dip in U_T) for 25 cycles <5 % U_T (>95 % dip in U_T) for 5 s	Mains power quality should be that of a typical commercial or hospital environment. If the user of the electric breast pump Swing maxi requires continued operation during power mains interruptions, it is recommended that the electric breast pump Swing maxi is powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

NOTE U_T is the a.c. mains voltage prior to application of the test level.

Electromagnetic Compatibility (EMC, IEC 60601-1-2:2007, Table 4)

Electromagnetic immunity			
The electric breast pump Swing maxi is intended for use in the electromagnetic environment specified below. The customer or the user of the electric breast pump Swing maxi should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	3 Vrms	Portable and mobile RF communications equipment should be used no closer to any part of the electric breast pump Swing maxi, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d = 1,2\sqrt{P}$
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2,5 GHz	3 V/m	$d = 1,2\sqrt{P}$ 80 MHz to 800 MHz $d = 2,3\sqrt{P}$ 800 MHz to 2,5 GHz where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey,^a should be less than the compliance level in each frequency range.^b Interference may occur in the vicinity of equipment marked with the following symbol: 

13. EMC/Technical description

Note 1 At 80 MHz and 800 MHz, the higher frequency range applies.

Note 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

^a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the electric breast pump Swing maxi is used exceeds the applicable RF compliance level above, the electric breast pump Swing maxi should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the electric breast pump Swing maxi.

^b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

Recommended separation distances between portable and mobile RF communications equipment and the electric breast pump Swing maxi

The electric breast pump Swing maxi is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the electric breast pump Swing maxi can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the electric breast pump Swing maxi as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter M		
	150 kHz to 80 MHz $d = 1,2\sqrt{P}$	80 MHz to 800 MHz $d = 1,2\sqrt{P}$	800 MHz to 2,5 GHz $d = 2,3\sqrt{P}$
0.01	0,12	0,12	0,23
0.1	0,37	0,37	0,74
1	1,2	1,2	2,3
10	3,7	3,7	7,4
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

Note 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

Note 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

14. Technical specifications

vacuum (approx.)
-20.....-270 mmHg
-3.....-35 kPa
45 ... 120 cpm

In	Out
100-240V~ 50/60 Hz	12V 1 A / 1.5 A

6 x 1.5V, Alkaline
AAA, Micro, LR03


121 x 129 x 57 mm


290 g

220 g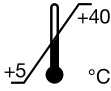

°C Operation

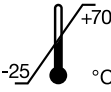
°C Transport / Storage

Operation

Transport / Storage

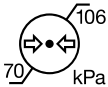Transport / Storage / Operation

 **Medela AG**
Lättichstrasse 4b
6341 Baar, Switzerland
www.medela.com

International Sales

Medela AG
Lättichstrasse 4b
6341 Baar
Switzerland
Phone +41 41 562 51 51
www.medela.com

OOO Medela

Vavilova str. 97
entrance 3, Office16–17
117335 Moscow
Russia
Phone +7 (495) 980 6194
Fax +7 (495) 980 6194
info@medela.ru
www.medela.ru