# Thanks for your order!

Get started with your Reusable Breast Milk Storage Bags.  
Best for home-use.

No leaks

Easy to  
clean & use

Durable

**One-piece, double seal,** ensuring no leaks for better protection of the unique nutrients and components naturally present in your precious milk.

**Double usage:** Use it to store in a fridge or in your freezer.

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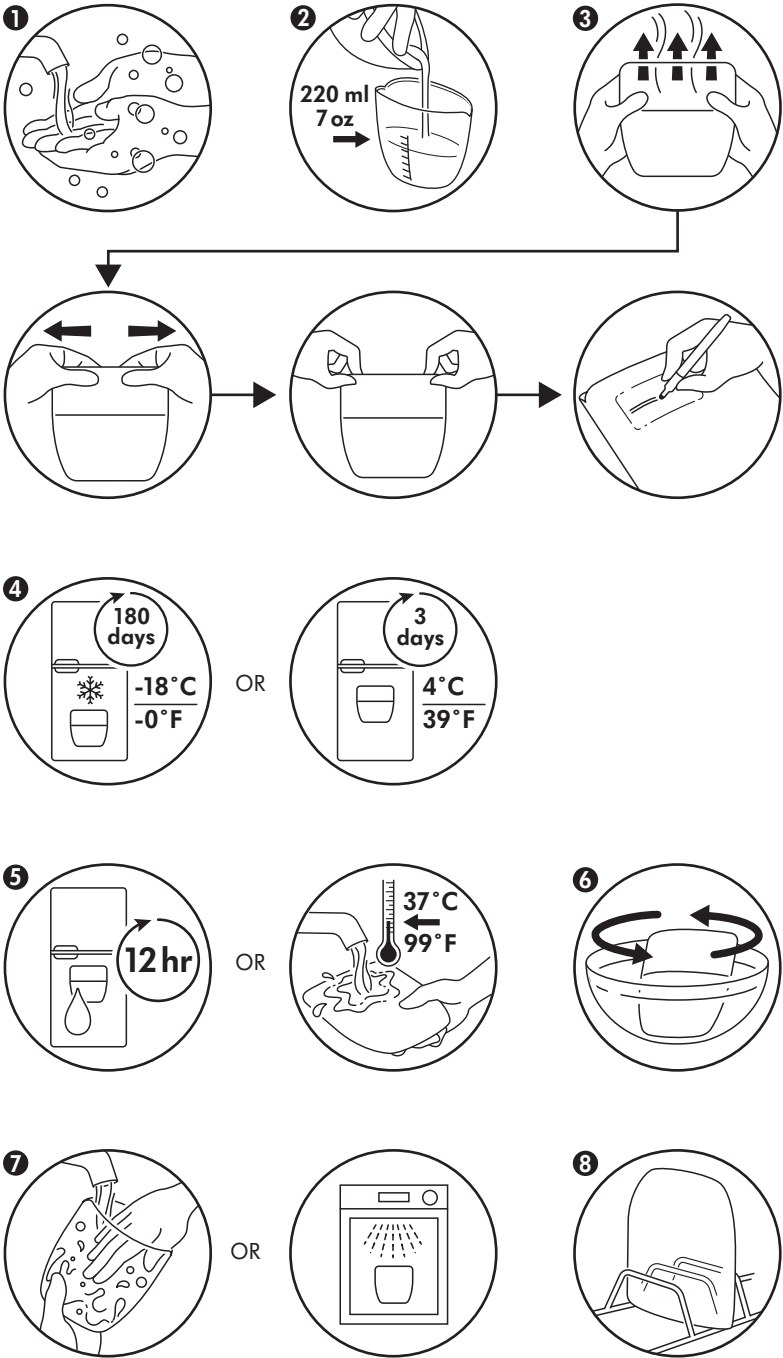
## ENGLISH

### Instructions for Use:

**Medela Reusable Breast Milk Storage Bags**  
**This bag is intended to be used for storing, transporting and warming up breast milk.**  
**Best for home-use.**

### Helpful Tips

- 1 Prior to first use, wash your hands with soap and clean the bag following instructions stated in 7.
- 2 Wash your hands with soap and water and dry thoroughly before touching the bag. Do not fill bag past the 220 ml / 7oz mark, breast milk expands as it freezes.
- 3 After filling, make sure the bag opening is clean and dry, gently remove the excess air from inside the bag and seal well. Seal and close the bag fully to avoid leakage. Label the breast milk storage bag before storing.
- 4 Refrigerate or freeze your breast milk immediately after expressing. Can be kept for 3 days in fridge or 180 days in the freezer. Avoid placing items > 1.2kg / 42oz on top of the bags when refrigerated or frozen.
- 5 To thaw, breast milk can be defrosted in the fridge in around 12 hours. Alternatively, hold the bag of frozen milk under warm running water (a maximum of 37 °C or 99 °F).
- 6 To warm your milk to body temperature (approx. 37°C or 99°F) by placing the storage bag into a bowl, jug or cup of lukewarm water. Gently swirl the storage bag during warming without shaking to mix the fat and other milk components.
- 7 Clean the bag immediately after use to avoid dry up of milk residues and to prevent from growth of bacteria. Rinse the bag in cold running water. Clean the bag using hand hot water with detergent, gently wipe the opening with a cloth, take care not to damage the bag when using a brush. Alternatively, you can wash the bags in the dishwasher upside down on the top rack (65°C or 149 °F). You can sanitize using the Quick Clean™ Microwave Bags or by boiling for 10 minutes (100°C or 212 °F).
- 8 After cleaning, allow the bag to air dry thoroughly in the upside-down position. After drying, seal it well and store it away until the next use. Avoid touching the inside of the bag.

**Material: Silicone**

**Disposal:** According to local regulations.

**Do not thaw frozen breast milk or warm breast milk in a microwave or a pan of boiling water to prevent burns.**

**Do not refreeze previously thawed breast milk.**

**Discard remaining breast milk after feeding the baby.**

**Each bag can be reused up to 500 times.**

**Keep the product away from sunlight. Food colorings may discolor components which will not affect performance.**

**Throw away at the first signs of damage or weakness.**

**Do not use a damaged product.**

**Contents may vary from picture.**

**Keep the product out of reach of children.**

**One-piece, double seal, ensuring no leaks for better protection of the unique nutrients and components naturally present in your precious milk.**

**Double usage:** Use it to store in a fridge or in your freezer.

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© Medela AG
Medela LLC
Medela Canada Inc.